


# Unleash 1000 uses with a single DNA Control

AMPLIRUN®

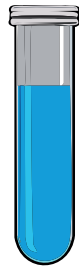


## DNA Control

Volume: 100  $\mu$ l  
Concentration range: 10,000-20,000 copies/ $\mu$ l

20  $\mu$ l neat control + 180  $\mu$ l TE buffer

## 1/10 diluted aliquots

Store at  
-20°C

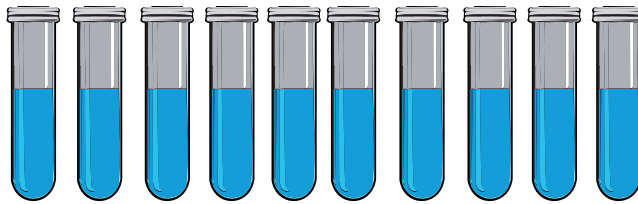

Volume per aliquot: 20  $\mu$ l  
Concentration: 1,000-2,000 copies/ $\mu$ l  
Used volumen from neat stock: 20  $\mu$ l



20  $\mu$ l diluted control + 180  $\mu$ l TE buffer



## 20 working aliquots

Store at  
-20°C

Volume per aliquot: 10  $\mu$ l  
Concentration: 100-200/ $\mu$ l