


Unleash 1000 uses with a single DNA Control

AMPLIRUN®

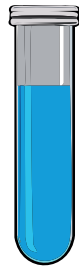

DNA Control


Volume: 100 μ l
Concentration range: 10,000-20,000 copies/ μ l

20 μ l neat control + 180 μ l TE buffer



1/10 diluted aliquots


Store at
-20°C

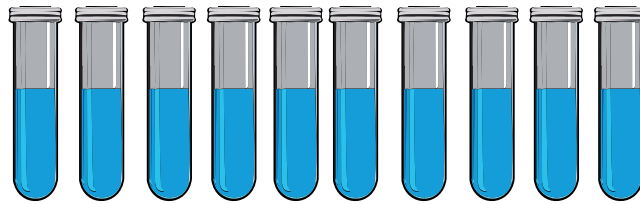

Volume per aliquot: 20 μ l
Concentration: 1,000-2,000 copies/ μ l
Used volume from neat stock: 20 μ l




20 μ l diluted control + 180 μ l TE buffer

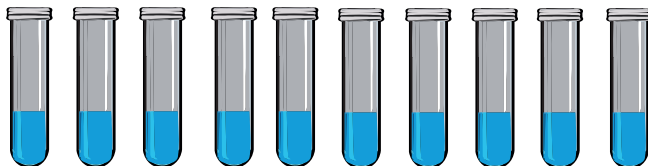

20 working aliquots

Store at
-20°C

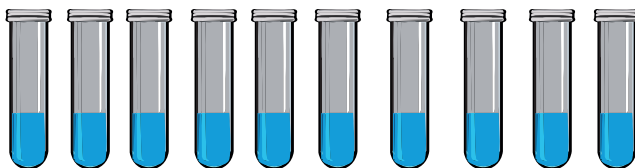

Volume per aliquot: 10 μ l
Concentration: 100-200 copies/ μ l