


Unleash 500 uses with one RNA Control

AMPLIRUN®

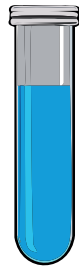


RNA Control

Volume: 50 μ l
Concentration range: 12,500-20,000 copies/ μ l

25 μ l neat control + 225 μ l TE buffer

1/10 diluted aliquots

Store at
-80°C

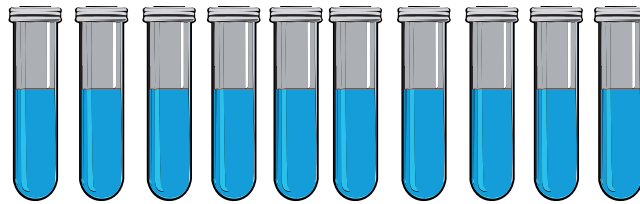

Volume per aliquot: 25 μ l
Concentration: 1,250-2,000 copies/ μ l
Used volume from neat stock: 25 μ l



25 μ l diluted control + 225 μ l TE buffer



25 working aliquots

Store at
-80°C

Volume per aliquot: 10 μ l
Concentration: 125-200 copies/ μ l